

# Fit-N-Swipe


## Cleaning Pads with Purpose

Available Through **DASH**  
MEDICAL GLOVES



- High cleaning effect and highly absorbent
- Practical, single-handed use
- Can be positioned individually in the direct working area
- Fit-N-Swipe Clean ideal for instruments
- Fit-N-Swipe Dry ideal for mirrors

### Can be placed anywhere!

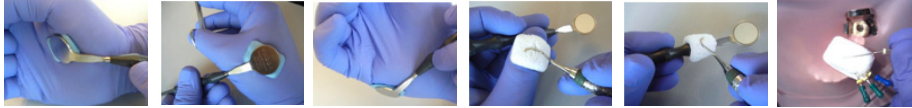

Ball of the thumb    The hand    Heel of the hand    The finger    The instrument    The rubber dam

## Fit-N-Swipe Clean

### Adhesive cleaning pad for clean instruments



- Unique microfiber with high retention capacity
- Safe and handy pad, use as pincushion for endodontic needles

605251    Fit-N-Swipe, Clean (white)    50 pads

DASH® Item #: FNSC



Cleaning scissors




Cleaning scaler



Use as a pincushion for endodontic needles

## Fit-N-Swipe Dry


### Adhesive drying pad for crystal clear mouth mirrors

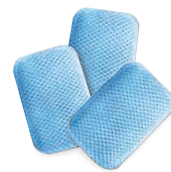


- Unique multi-layered fabric
- Removes even greasy or oily contaminants without streaking

605252    Fit-N-Swipe, Dry (blue)    50 pads

DASH® Item #: FNSM

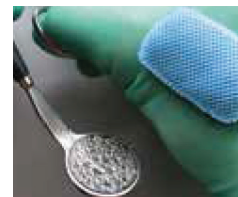

Wet and fogged mouth mirror

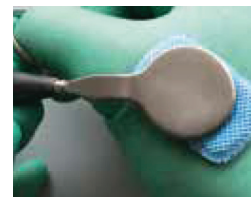

Simply wipe off on the pad



Crystal clear mouth mirror

### Testimonials

**Bettina Drewes, DH:** "I think Fit-N-Swipe is a very good product as there usually is no assistant available in prophylactic treatment and Fit-N-Swipe makes the work so much easier."

**Melanie Graf, ZMP:** "Absolutely great! It's simple and easy. 'Peel off and stick on the glove - ready!' After the treatment simply dispose of the glove."

\* Can be sterilized in autoclave before use

Check out the video



Manufactured By:



www.hagerworldwide.com | (800) 328-2335



**Contact Us**  
Call: 800-523-2055  
Text: 414-421-2229  
Email: dash@dashmedical.com

# Fit-N-Swipe

## Instructions for use

### Fit-N-Swipe Clean

#### Adhesive pads to help clean instruments during use

- Convenient single-handed use
- Unique microfiber fabric with high retention capacity
- High cleaning effect and highly absorbent
- Versatile placement options
- Pads are single-use and non-sterile. They can be sterilized prior to the single-use in an autoclave according to the instructions for use for textile fabrics.

#### Positioning the Pads

- Bend the pad for an easier removal of silicone paper
- Press the pad firmly. When changing position a new pad must be used
- All surfaces (glove, rubber dam) must be absolutely dry
- Any residue of disinfectant or soap must be removed
- Do not stick the pad directly to the skin and do not reuse it

#### Adhesion

Adhesion tested on vinyl, nitrile, latex. Special glove coatings can reduce the adhesiveness.



Bend the pad for an easier removal of silicone paper

### Fit-N-Swipe Dry

#### Adhesive drying pad for crystal clear mouth mirrors

- Unique multi-layer fabric
- High cleaning effect and highly absorbent
- Dries mirror streak-free
- Versatile placement options
- Pads are single-use and non-sterile. They can be sterilized prior to the single-use in an autoclave according to the instructions for use for textile fabrics.

#### Positioning the Pads

- Press the pad firmly. When changing position a new pad must be used
- All surfaces (glove, rubber dam) must be absolutely dry
- Any residue of disinfectant or soap must be removed
- Do not stick the pad directly to the skin and do not reuse it

#### Adhesion

Adhesion tested on vinyl, nitrile, latex. Special glove coatings can reduce the adhesiveness.



Bend the pad for an easier removal of silicone paper

Manufactured By:



www.hagerworldwide.com | (800) 328-2335



#### Contact Us

Call: 800-523-2055

Text: 414-421-2229

Email: [dash@dashmedical.com](mailto:dash@dashmedical.com)